


Eszközök

- Steril szike, tárgylemez/gézlap
- Steril vattapálca (mellékelve)
- Sabouraud-féle folyékony médium klóramfenikollal (mellékelve)




1.

Sterilizált szikével kaparékot veszünk a köröm alól tárgylemezre vagy gézlapra.

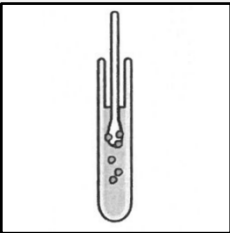

2.

A kaparékot a vattapálcával összeszedjük és belemossuk a Sabouraud médiumot tartalmazó kémcsőbe.

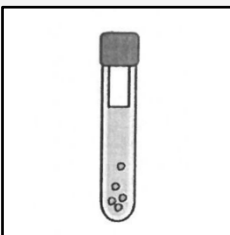


3.

Az így elkészített körömkaparékkal beoltott médiumot eredeti kupakjával lezárjuk.

A mintát betegazonosítóval látjuk el, szobahőmérsékleten szállítjuk a laboratóriumba feldolgozásra.


A mintavevőt felhasználás előtt és után is álló helyzetben szükséges tárolni, illetve a mintavevő szállítása is ebben a helyzetben kell, hogy megtörténjen.

NAIL SCRAPING SAMPLE COLLECTION

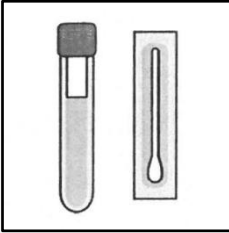


Equipment

- Sterile scalpel, microscope slide / gauze pad
- Sterile cotton swab (included)
- Sabouraud liquid medium with chloramphenicol (included)

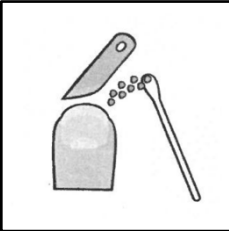


1.

Using a sterile scalpel, a scraping sample is collected from beneath the nail onto a microscope slide or gauze pad.

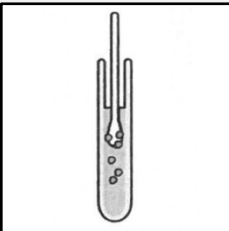

2.

The scraping sample is collected with the cotton swab and transferred into the test tube containing the Sabouraud medium.

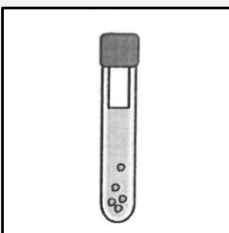

3.

The medium inoculated with the prepared nail scraping specimen is sealed with its original cap.

The sample is labeled with the patient identifier and transported to the laboratory at room temperature for processing.

The sampler must be stored in an upright position both before and after use, and it must also be transported this way.



MEDICARE
DIAGNOSTIC CENTER